



**PLANET LIGHTING.
MADE IN AUSTRALIA.
EXPORTED TO THE
WORLD.**

**CUSTOMER SERVICE:
+ 61 (0)2 6659 5600
MONDAY - FRIDAY
9AM - 5PM**

12 Tamarind Drive Bellingen
NSW 2454 Australia
Phone + 61 (0)2 6659 5600
Fax + 61 (0)2 66552121
info@planetlighting.com
planetlighting.com



**ULED EXAMINATION LIGHT TROLLEY MOUNT
ASSEMBLY INSTRUCTIONS**

22.0420

WARNINGS AND SAFETY NOTICES

DANGER – RISK OF ELECTRIC SHOCK

DO NOT assemble product with the power cable connected to power. Ensure plug is disconnected from mains socket.

WARNING – READ MANUAL BEFORE USE

Read the user manual before use and observe safe operation of product at all times. Contact your supplier or the manufacturer if you have any questions.


WARNING – INDOOR USE ONLY

Product is designed for indoor use only.

TOOLS REQUIRED



M4 Allen key

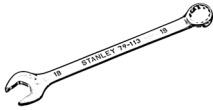

18mm spanner / socket

4

Insert ULED head jack (A) into arm then depress lock / quick release button to secure head onto arm.

Note, the lock / quick release button is provided for head removal and maintenance

3

Connect plug (A), then slide upper trolley tube (B) onto the spigot in the bottom tube (C). Secure with the supplied M6 x 10 button head socket (D) and tighten with an M4 Allen key.

2

Place bottom trolley tube (A) onto mobile base (B). Secure with the supplied bolt and washer (C), using an 18mm spanner/socket to tighten.

Note, perform this step on a benchtop for easier access to base.

1

Push castor wheels into base.

Note, ensure castors are securely fastened.



MADE IN AUSTRALIA.
EXPORTED TO THE
WORLD.

CUSTOMER SERVICE:
+ 61 (0)2 6659 5600
MONDAY - FRIDAY
9AM - 5PM

**PLANET
MEDICAL**

12 Tamarind Drive Bellingen
NSW 2454 Australia
Phone + 61 (0)2 6659 5600
Fax + 61 (0)2 66552121
info@planetlighting.com
planetlighting.com