




PLANET LIGHTING.  
MADE IN AUSTRALIA.  
EXPORTED TO THE  
WORLD.

**CUSTOMER SERVICE:**  
**+ 61 (0)2 6659 5600**  
**MONDAY - FRIDAY**  
**9AM - 5PM**

12 Tamarind Drive Bellingen  
NSW 2454 Australia  
Phone + 61 (0)2 6659 5600  
Fax + 61 (0)2 66552121  
[info@planetlighting.com](mailto:info@planetlighting.com)  
[planetlighting.com](http://planetlighting.com)

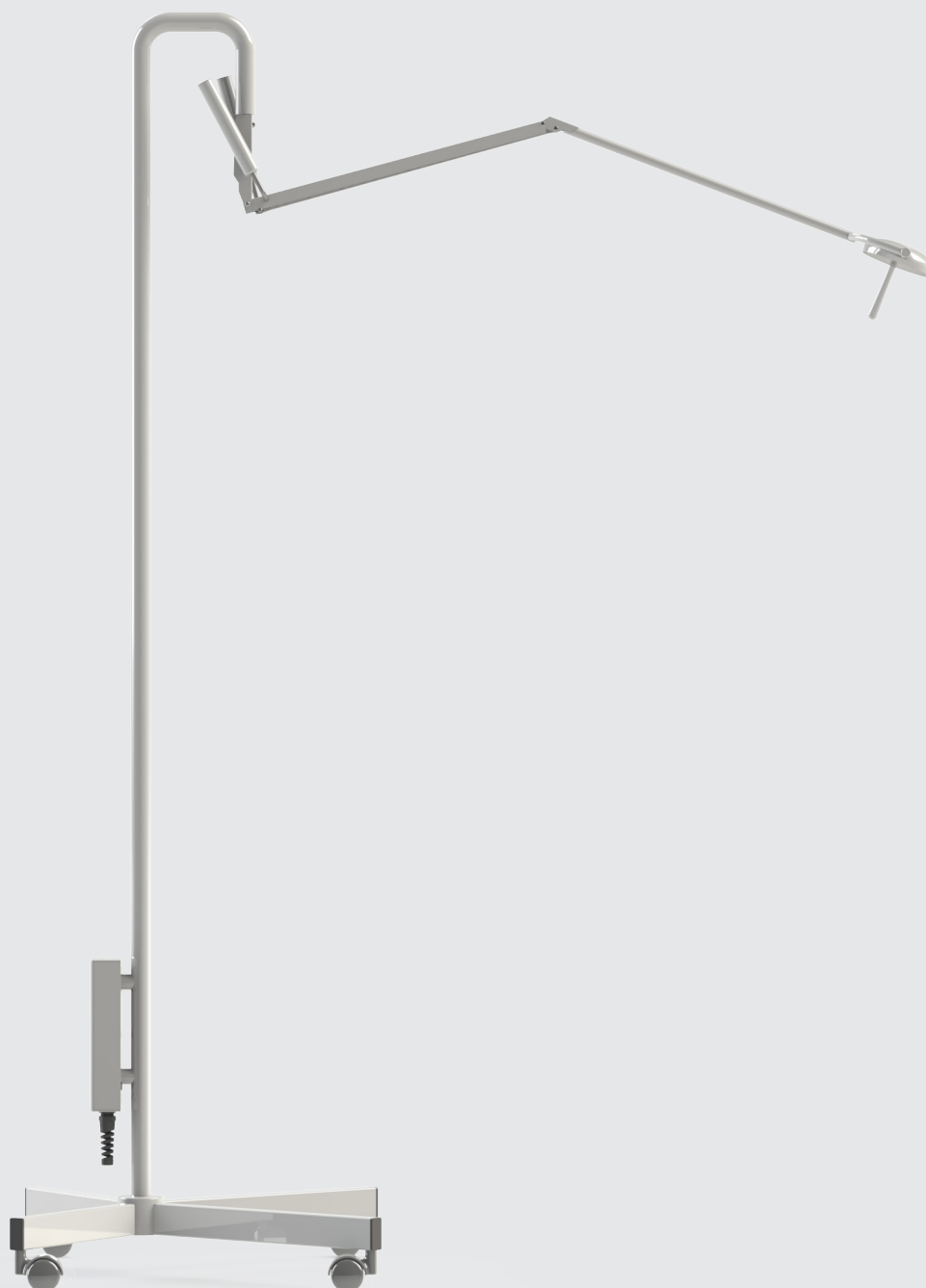

**ULED EXAMINATION LIGHT TROLLEY MOUNT  
ASSEMBLY INSTRUCTIONS**

**22.0420**

## WARNINGS AND SAFETY NOTICES

### **DANGER – RISK OF ELECTRIC SHOCK**

DO NOT assemble product with the power cable connected to power. Ensure plug is disconnected from mains socket.

### **WARNING – READ MANUAL BEFORE USE**

Read the user manual before use and observe safe operation of product at all times. Contact your supplier or the manufacturer if you have any questions.


### **WARNING – INDOOR USE ONLY**

Product is designed for indoor use only.

## TOOLS REQUIRED



M4 Allen key

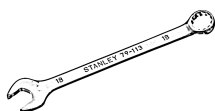

18mm spanner / socket

4

Insert ULED head jack (A) into arm then depress lock / quick release button to secure head onto arm.

*Note, the lock / quick release button is provided for head removal and maintenance*

3

Connect plug (A), then slide upper trolley tube (B) onto the spigot in the bottom tube (C). Secure with the supplied M6 x 10 button head socket (D) and tighten with an M4 Allen key.

2

Place bottom trolley tube (A) onto mobile base (B). Secure with the supplied bolt and washer (C), using an 18mm spanner/socket to tighten.

*Note, perform this step on a benchtop for easier access to base.*

1

Push castor wheels into base.

*Note, ensure castors are securely fastened.*

MADE IN AUSTRALIA.  
EXPORTED TO THE  
WORLD.

CUSTOMER SERVICE:  
+ 61 (0)2 6659 5600  
MONDAY - FRIDAY  
9AM - 5PM

**PLANET**  
**MEDICAL**

12 Tamarind Drive Bellingen  
NSW 2454 Australia  
Phone + 61 (0)2 6659 5600  
Fax + 61 (0)2 66552121  
info@planetlighting.com  
planetlighting.com

